

## ComfortFusion™

INSTRUCTIONS FOR USE | MODE D'EMPLOI | INSTRUCCIONES DE USO  
GEBRAUCHSANWEISUNG | ISTRUZIONI PER L'USO | BRUKSANVISNING  
BRUGSANVISNING | GEBRUIKSAANWIJZING | KÄYTTÖOHJEET  
BRUKSANVISNING | INSTRUÇÕES DE UTILIZAÇÃO | INSTRUÇÕES PARA UTILIZAÇÃO  
ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ | INSTRUKCJA STOSOWANIA | NÁVOD K POUŽITÍ  
HASZNÁLATI UTASÍTÁS | NÁVOD NA POUŽITIE | NAVODILA ZA UPORABO  
使用说明 | KULLANIM TALIMATLARI

## Removing the Headgear and Mask

First remove one (or both) Quick Clips from the mask sockets. Grasp the body of the Quick Clip and gently pull it out of the socket away from the face. After the Quick Clip is disengaged, slide the mask over your head.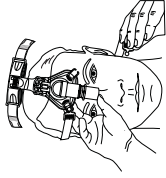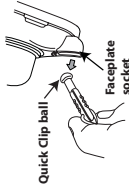


## The Retaining Ring and Cushion Assembly

To remove:


Unhook the retaining clips on the bottom and the sides of the mask. **A)** Remove the retaining ring. **B)** Remove the cushion.

To attach:

Note the cutouts on the bottom of the cushion and the alignment marks on the bottom of the cushion, faceplate and retaining ring. **B)** Place the cushion against the faceplate. **A)** Put the retaining ring around the cushion. Press the ring firmly against the faceplate until all three sides click into place. Gently tug on the cushion and the ring to make sure all of the parts are securely in place.

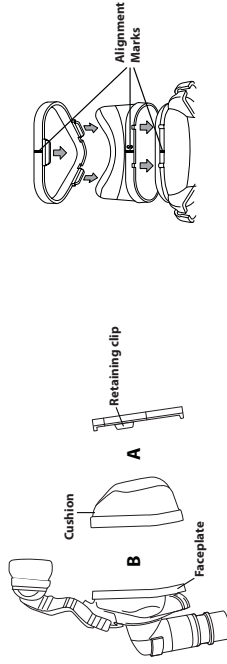## Intentional Leak
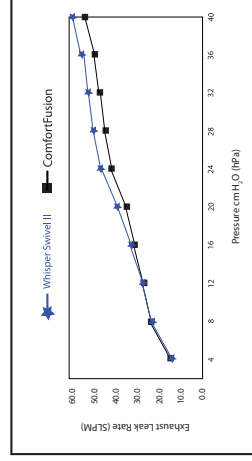
## Deadspace Volume

S 130 ml  
M 140 ml

## Pressure Drop cm H<sub>2</sub>O (hPa)

50 SLPM 100 SLPM  
S 0.1 0.4  
M 0.1 0.4

## Storage Conditions:

-20°C to +60°C  
15% to 95% RH, non-condensing

**MULTI-PATIENT USE REUSABLE**

**LATEX FREE**

## Intended Use

The ComfortFusion Nasal Mask is intended to provide an interface for Continuous Positive Airway Pressure (CPAP) or bi-level therapy. The mask is for multi-patient use on patients (>30 kg) in the home or hospital/institution environment.

An exhalation port is built into the mask so that a separate exhalation port is not required.

## Symbols

General warning or caution

Hand wash

Consult accompanying documents

## Warnings

- This mask is designed for use with CPAP or bi-level systems recommended by your health care professional or respiratory therapist. Do not wear this mask unless the CPAP or bi-level system is turned on and operating properly. Do not block or try to seal the exhalation port. **Explanation of the Warning:** CPAP systems are intended to be used with special masks with connectors which have vent holes to allow continuous flow of air out of the mask. When the CPAP machine is turned on and functioning properly, new air from the CPAP machine flushes the exhaled air out through the attached mask exhalation port. However, when the CPAP machine is not operating, enough fresh air will not be provided through the mask, and exhaled air may be rebreathed. This warning applies to most models of CPAP systems.
- This warning applies to most CPAP devices. At low CPAP pressures, the airflow through the exhalation port may not be enough to clear all of the exhaled gas (CO<sub>2</sub>) from the mask. You may breathe in some of the air that you have exhaled.
- If oxygen is used with the CPAP machine, the oxygen flow must be turned off when the CPAP machine is not operating. **Explanation of the Warning:** When the CPAP device is not in operation, and the oxygen flow is left on, oxygen delivered into the ventilator tubing may accumulate within the CPAP machine enclosure. Oxygen accumulated in the CPAP machine enclosure will create a risk of fire. This warning applies to most types of CPAP machines.
- Oxygen supports combustion. Oxygen should not be used while smoking or in the presence of an open flame.
- At a fixed flow rate of supplemental oxygen flow, the inhaled oxygen concentration will vary, depending on the pressure settings, patient breathing pattern, nasal interface selection, and the leak rate. This warning applies to most types of CPAP machines.
- Some users may experience skin redness. If this happens, contact your health care professional.

## Disposal

Dispose of in accordance with local regulations.

## Before Use

- Hand wash mask.
- Read and understand the instructions completely.
- Remove the Quick Fit Tips from the headgear.
- Clean your face before wearing the mask.

## Cleaning Instructions

**For Multi-Patient Use:** Visit [www.respironics.com](http://www.respironics.com) or contact Respironics at 1-724-387-4000 or Respironics Deutschland at +49 8152 93060 to obtain mask disinfecting instructions.

**For Home Use:** The mask and tubing should be cleaned daily. The headgear should be cleaned weekly or as needed.

## Clean the Mask

1. Hand wash in warm water with a mild liquid dishwashing detergent.  
**Do not** use bleach, alcohol, or cleaning solutions containing bleaching agents or alcohol.  
**Do not** use cleaners containing conditioners or moisturizers.
2. Rinse thoroughly. Pat with a towel and air dry completely before use.
3. Inspect all parts for damage or wear; replace if necessary.

## Cleaning the Headgear

The headgear should be cleaned weekly or as needed, either by hand or machine wash in the gentle cycle. Line dry the headgear.

## Dishwasher Instructions

1. Remove the headgear. **Do not** wash the headgear in the dishwasher.
2. Separate the cushion and the faceplate and wash in the top shelf of the dishwasher.

**CAUTION:** Use liquid dishwashing detergent only.

## Removing the Headgear from the Mask

Please use one of the following methods to remove the headgear from the mask.

- Method 1 Undo the four headgear tabs on the headgear and pull them through the slots in the forehead support bracket and on the Quick Clips.
- Method 2 Disconnect the Quick Clips and forehead support arm. Pull the Quick Clip ball up and away from the socket. Do not pull the ball through the socket.


Next, disconnect the forehead support pad from the forehead support arm by pressing in on the center of the forehead support bracket and then squeezing the upper sides of the support arm together to disengage the forehead support bracket from the forehead support arm.

## Attaching the Headgear to the Mask

1. To reattach, thread the top headgear straps through the slots in the forehead support bracket and fold the headgear back so the tabs will catch the headgear straps.
2. Attach the Quick Clips by placing the ball into the socket and gently pressing into place. Then connect the forehead support arm by pressing in on the center of the forehead support bracket and then squeezing the upper sides of the support arm together and sliding the tabs into the forehead support pad until they latch into place.


## Getting the Right Cushion Size

Identify the areas of your face: **A)** sides of the nostrils, **B)** bridge of the nose zone, and **C)** between the nose and the upper lip. Use the sizing gauge that is integrated into the product bag to get the right size cushion. Choose the smallest mask size that will cover these facial areas.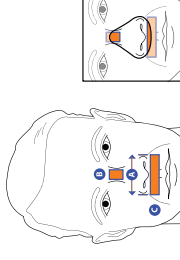

## Achieving the Right Fit

1. Open the headgear to a large setting.
2. Adjust the forehead arm to the highest position. To adjust, gently squeeze the arms of the StabilitySelector and slide the arm upwards.
3. Disconnect one, or both of the Quick Clips. Simply hold the mask cushion gently over your nose and pull the headgear on over your head. Connect the Quick Clip if detached.
4. Adjust the top headgear strap using both hands and move the headgear tabs to a comfortable position. The forehead pad should be touching the forehead lightly. Adjust the bottom strap so that the mask is sitting gently on the face.
5. Connect the flexible tubing (included with your CPAP or bi-level device) to the swivel elbow on the mask.
6. Turn on the airflow.
7. Lie down and breathe normally through your nose, keeping your lips closed. If the forehead support does not rest comfortably against your forehead, tighten the top headgear strap.



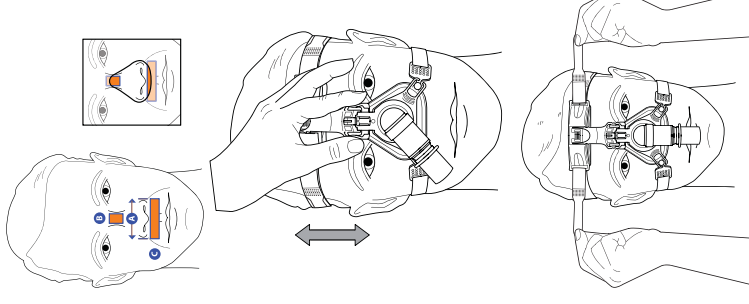
## Troubleshooting Leaks

You may need to adjust the forehead support arm to reduce leaks or pressure points at the bridge of the nose or above the lip. We offer these tips to adjust for leaks:

- For leaks around the bridge of the nose or into the eyes gently squeeze the arms of the StabilitySelector and adjust to a lower position.
- For leaks under the nose or near the lip gently squeeze the arms of the StabilitySelector and adjust to a higher position.
- Reseat the mask by pulling it directly away from the face and gently setting it back into place.

The most common mistake is overtightening the headgear. The headgear should fit loose and the mask and headgear should feel comfortable when worn.

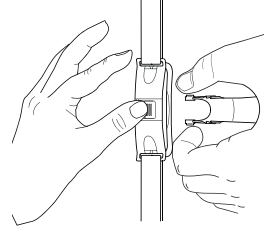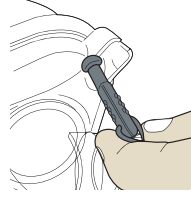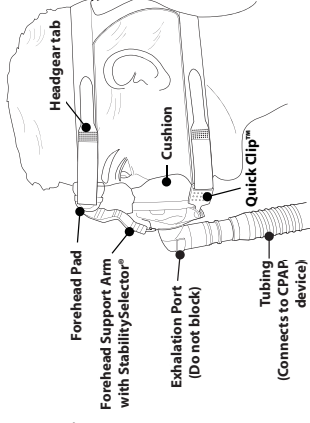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