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Legende

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1 Device Description

Key

- 1 Forehead cushion
- 2 Mask seal
- 3 Head strap
- 4 Rotating sleeve
- 5 Support
- 6 Seal for ports
- 7 Mask body
- 8 Elbow
- 9 Exhalation connector
- 10 Exhalation slot
- 11 Hose connector

Purpose of Use

The nasal mask with exhalation connector is an accessory that serves as a sealing element between patient and therapy device. It is used in the treatment of sleep apnea and for non-invasive ventilation of patients with respiratory insufficiency.

Functional Description

The nasal mask and the head strap are attached over your nose so that the flow of air from the therapy device is routed through the nasal mask to the nose. The form and flexible material of the mask seal creates an airtight bond to the face. The elbow connector allows the greatest possible freedom in positioning the tubing.

The inhaled air flows through the mask/hose system. The exhaled air escapes through a valve between the elbow and the rotating sleeve. This allows you to breathe quietly during therapy. The exhalation connector avoids the risk of re-inhalation.

2 For Your Safety

Safety regulations

For your and your patients' safety and in accordance with the requirements of the European Union Guidelines 93/42/EEC, please observe the following points:

- Please read these instructions for use very carefully. They are a part of the mask and should be available at all times.
- Modifications and a lack of biocompatibility may result if third party articles are used which are not listed in these cases any warranty entitlement and liability shall lapse where the accessories recommended in the instructions for use or original spare parts are not utilized.
- The mask is designed for therapy pressure between 4 and 30 HfA. The nasal mask must not be used at higher or lower therapy pressures.

Note that patients with limited spontaneous respiration must be monitored at all times and that the appropriate negative pressure/leak alarm on the ventilation device must be activated.

Patients with obstructive and restrictive lung diseases should be examined on an individual basis during use of the SOMNOpus®, for example with regard to re-inhalation of CO₂.

The SOMNOpus® has an integrated exhalation system. Do not connect any other exhalation system or exhalation valve to the nasal mask.

Do not block any of the openings in the nasal mask.

Only put the nasal mask over your face for a prolonged period when the therapy device is switched on. The flow of air from the therapy device serves among other things to remove the exhaled air from mask and hose. If the therapy device is not switched on, there is a risk of re-inhalation when you breathe through your nose.

Do not use a breathing hose that is electrically conductive.

Clean the mask as described under Section 4 "Hygienic preparation" / "Inappropriate cleaning procedures, unsuitable cleaning agents and infection agents can lead to an infection or bacterial contamination and can impede the functioning of the mask."

Side effects

The following side effects may occur during short-term or long-term use: nasal congestion, dry nose, dry mouth in the morning, sinus pressure, irritation of the conjunctiva as well as irritating noises during exhalation. Wearing the mask can also lead to tenderness of the face or reddening of the skin.

3 Operation

Assemble and adjust forehead support

The forehead support is made up of a support and a forehead cushion. The support makes it possible to adjust the mask to your face while the forehead cushion allows for comfortable distribution of the mask pressure across the forehead. By adjusting the support, pressure on the bridge of the nose can be minimized (Fig. A).

Select the support size that best suits your face (Fig. B). Attach the forehead cushion to the support and place the support in the slit on the mask body. In most cases, the size 3 support will be appropriate. The forehead cushion is already set up on this support when you receive the mask.

Place the mask on your face and check to see if you have found the correct support.

If the mask isn't tight or if it causes uncomfortable tenderness, shift the support within the slot of the mask or choose a different support size.

Dismantle forehead support

Loosen the forehead support by pressing on the clamp on the mask body (Fig. C). Pull the forehead cushion off support.

Head strap adjustment

When you first put on the nasal mask with the head straps, it is a good idea to ask another person for help.

Thread the two Velcro straps of the head strap into the slots on the mask body. You can use either the lateral slots or the top hoops to attach the Velcro forehead straps. This is not necessary the first time the nasal mask is put on, as the SOMNOpus® is already fitted.

Pass the head strap over your head and hold the nasal mask over your nose.

Adjust the head and neck straps of the head strap so that the mask sits firmly, but not too tightly, against your face (Fig. D).

Connection to other devices

Push the hose of your therapy device onto the rotating sleeve (Fig. E). In doing so, ensure that there is a secure, airtight connection to the hose. The connection should not come loose as a result of therapy pressure.

If you wish to interrupt therapy briefly, you can pull the rotating sleeve out of the elbow. It is not necessary to remove the mask or pull the breathing hose off the rotating sleeve with this nasal mask.

Combining with other devices

This nasal mask can be combined with any device to perform the therapies described under "Purpose of use", as long as the following conditions are met:

- when combining the nasal mask with other devices, be aware that the accessories recommended in the instructions for use are used.
- when combining the nasal mask with other devices, be aware that the correct therapy pressure can be achieved in the mask using the desired combination of devices.
- when combining the nasal mask with other devices, be aware that the correct therapy pressure can be achieved in the mask using the desired combination of devices.

Take account of the specifications in the instructions for use for the devices you intend combining with the nasal mask.

Use with cold-air or warm-air humidifier

The nasal mask system can be used with a cold-air or warm-air humidifier. The degree of humidity you choose should not be so high that water condenses in the mask or hose. Please be sure to read the instructions for use for your humidifier and your therapy device.

Pressure measurement

SOMNOpus® is supplied with two ports. They can be used for external pressure measurement.

If only one port is required, turn the connection upwards (Fig. F). When the functions with the ports are no longer required, they should be made airtight with the enclosed seal. Otherwise air can escape through the ports and internal pressure will not be maintained.

After use

- Undo one Velcro strap on the head strap and remove the mask.
- Clean the mask as described in Section 4.

Dismantling the mask

1. Remove the head strap from the nasal mask.
2. Loosen the forehead support from the mask body by pressing on the clamp. Pull the forehead cushion off the support.
3. Pull the mask seal off the mask body and remove the seal for the ports.
4. Pull the rotating sleeve out of the elbow and remove the elbow from the mask body by pressing gently inward (Fig. G).

Assembling the mask

1. Push the elbow into the mask body from inside. Push the rotating sleeve into the elbow. Make sure both parts engage properly.
2. Push the mask seal onto the mask body and the port seal onto the pressure measuring port.
3. Attach the forehead support to the mask body as described under "Assemble and adjust forehead support".
4. Attach the head strap to the nasal mask as described under "Head strap adjustment".

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Weekly cleaning, disinfecting or sterilizing

The nasal mask should be cleaned thoroughly at least once per week. For cleaning purposes, take the mask apart as described above.

4.2 Procedure for cleaning, disinfecting or sterilizing

The following procedures are allowed:

Procedures	Procedures	Procedures	Procedures	Procedures	Procedures	Procedures	Procedures	Procedures	Procedures
	Cleaning by hand	Disinfection	Sterilization (in an autoclave)		Disinfection	Sterilization (in an autoclave)		Disinfection	Sterilization (in an autoclave)
Mask seal	X	X	X	X	X	X	X	X	X
Forehead cushion	X	X	X	X	X	X	X	X	X
Seal for ports	X	X	X	X	X	X	X	X	X
Mask body	X	X	X	X	X	X	X	X	X
Support	X	X	X	X	X	X	X	X	X
Elbow	X	X	X	X	X	X	X	X	X
Rotating sleeve	X	X	X	X	X	X	X	X	X
Head strap	X	X	X	X	X	X	X	X	X

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I

NL

Table with 2 columns: Cleaning/Disinfection method and instructions in German.

Table with 2 columns: Cleaning/Disinfection method and instructions in English.

Table with 2 columns: Cleaning/Disinfection method and instructions in French.

Table with 2 columns: Cleaning/Disinfection method and instructions in Italian.

Table with 2 columns: Cleaning/Disinfection method and instructions in Dutch.

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4.3 Instructions for Disinfecting and Sterilizing

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For patients with a weakened immune system or a special medical history daily disinfection or sterilization may be necessary as for a change in patients.

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4.4 Disinfecting and Sterilizing for new patient

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Il nasal mask is to be used for another patient, it must be subjected to hygiene measures first.

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Please carry out a visual check once per week.

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If the mask seal is no longer airtight due to a tear, please change it.

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Troubleshooting table with columns: Fault, Cause, Remedy.

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Supply schedule table with columns: Supply Schedule, consisting of, Mask seal, etc.

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8 Technische gegevens

Technical data table with columns: Geräteklasse, Abmessungen, Gewicht, etc.

Technical data table with columns: Device class, Dimensions, Weight, etc.

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Replacement parts table with columns: Replacement parts, Forehead cushion, Support frontal, etc.

Replacement parts table with columns: Pièces de rechange, Coussinet frontal, Support frontal, etc.

Replacement parts table with columns: Ricambi, Cuscinetto per la fronte, Appoggio frontale, etc.

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All individual parts of the nasal mask are latex-free.

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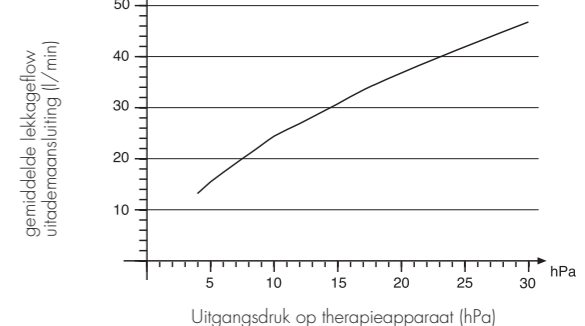
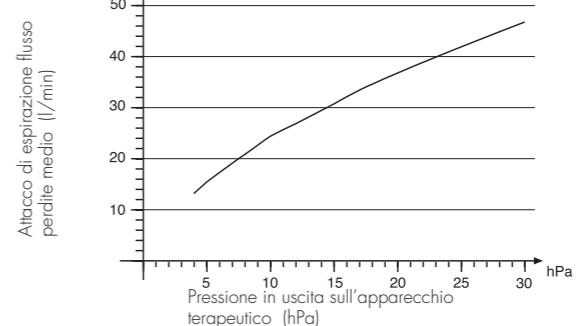
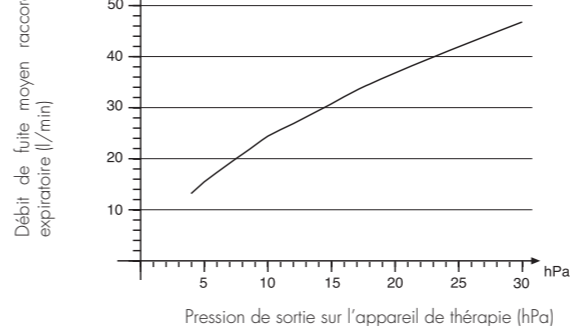
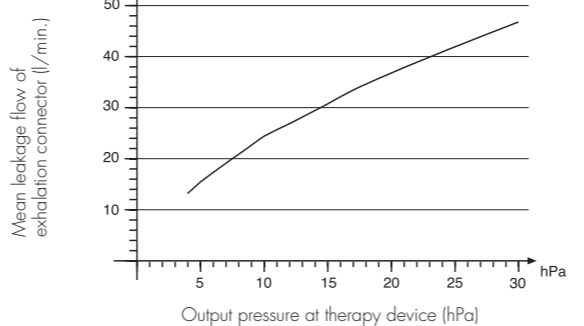
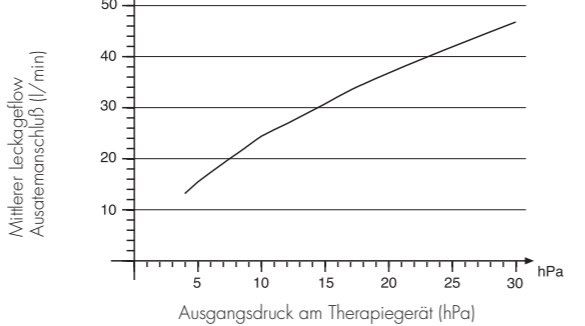
Druck-Flow-Kennlinie

Characteristic pressure/flow graph

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Weinmann guarantees that the product, when used in accordance with requirements, will remain free from defects for a period of 6 months from date of purchase.

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WEINMANN logo and contact information.

WEINMANN logo and contact information.

WEINMANN logo and contact information.

WEINMANN logo and contact information.

WEINMANN logo and contact information.